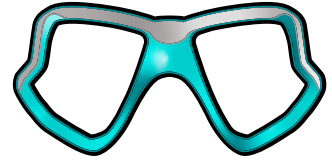


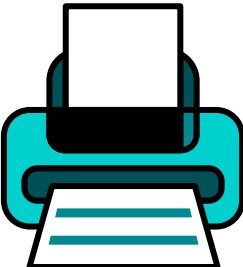
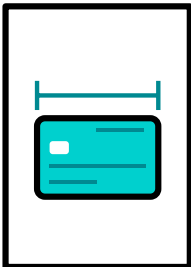

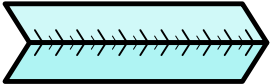


DIVESIGHT™

MEASURE YOUR PUPILLARY DISTANCE WITH OUR HANDY PRINTABLE MEASURING SYSTEM

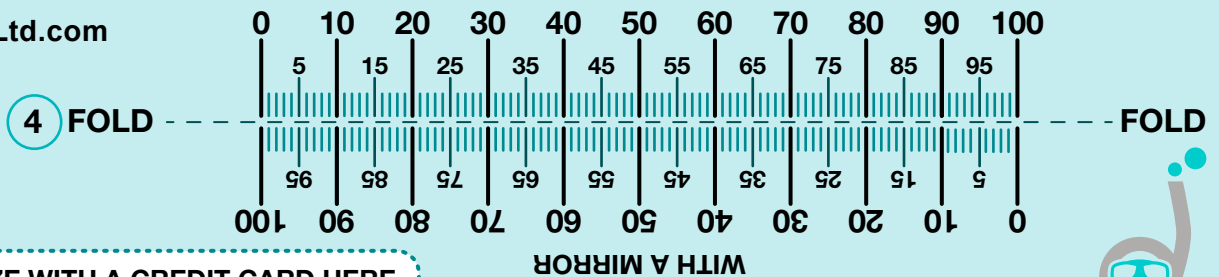
The measurement between your pupils is called 'the pupillary distance' and is used when customising lenses for your mask. Use our printable measuring tool to establish your own PD.



- 
Print this sheet out at 100% or 'actual size'. Ensure the printer settings are not set to scale the content in any way.
- 
Once printed, align the longest edge of a credit card on the guide below. If it does not fit, do not use the sheet and contact us instead.
- 
Cut out the measuring tool along the dotted line.
- 
Fold the measuring tool along the fold line.

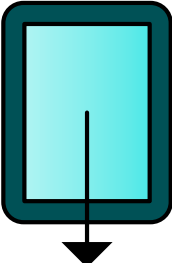
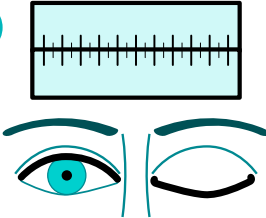
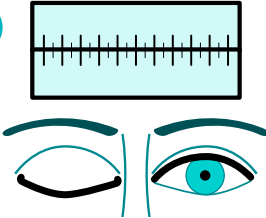
T: +44(0)114 327 3140
E: info@DiveSightLtd.com
www.DiveSightLtd.com

WITH HELP FROM A FRIEND



2 MATCH FOR SIZE WITH A CREDIT CARD HERE

USING THE MEASURING TOOL

- 
Stand or sit in front of a mirror approximately 8 inches / 200mm away, or at arms length from a friend.
- 
Close your left eye and place the 0 directly over the pupil of the open eye.
- 
Close your right eye and read the number directly over the centre of your left pupil.

THIS NUMBER IS YOUR PUPILLARY DISTANCE (PD)